

**Orthopaedic Suction Cannulas and Orthopaedic Suction Sets
Instructions for use**

Ref no.: 0202-XXXCKFT, 0202-30007SSXCKFT

 <p>GRENA[®] Think Medical Grena (Qingdao) Medical Devices Ltd., No 318 Huanghe West Road, Huangdao District, Qingdao City, Shandong Province, 266555, China</p>	<p>Contact information: Phone: +86 0532-86769190 Fax: +86 0532-86767322</p>	<table border="1"> <tr> <td>EC</td> <td>REP</td> </tr> </table> <p>MDML INTL Limited, 10 McCurtain Hill Clonakilty, Co. Cork Republic of Ireland</p>	EC	REP	
EC	REP				

Indications:

Orthopaedic suction cannulas and orthopaedic suction sets are used to draw liquids from the patient during orthopaedic surgery procedures when bone crumbs can be expected in surgical field.


Contraindications:

Orthopaedic suction sets and suction cannulas should not be used if delicate tissue can be damaged by suction.

Description of the device:

Orthopaedic suction sets and suction cannulas are sterile and disposable. They are made of medical grade raw materials. Cannula can be easily disconnected from the suction tube and replaced by a new one if necessary. Built-in filter retains bone crumbs inside the cannula allowing free fluid passage. Filter can be replaced if necessary. Replaceable tips facilitate cannula adaptation for any particular procedure conditions.

Instructions for use:

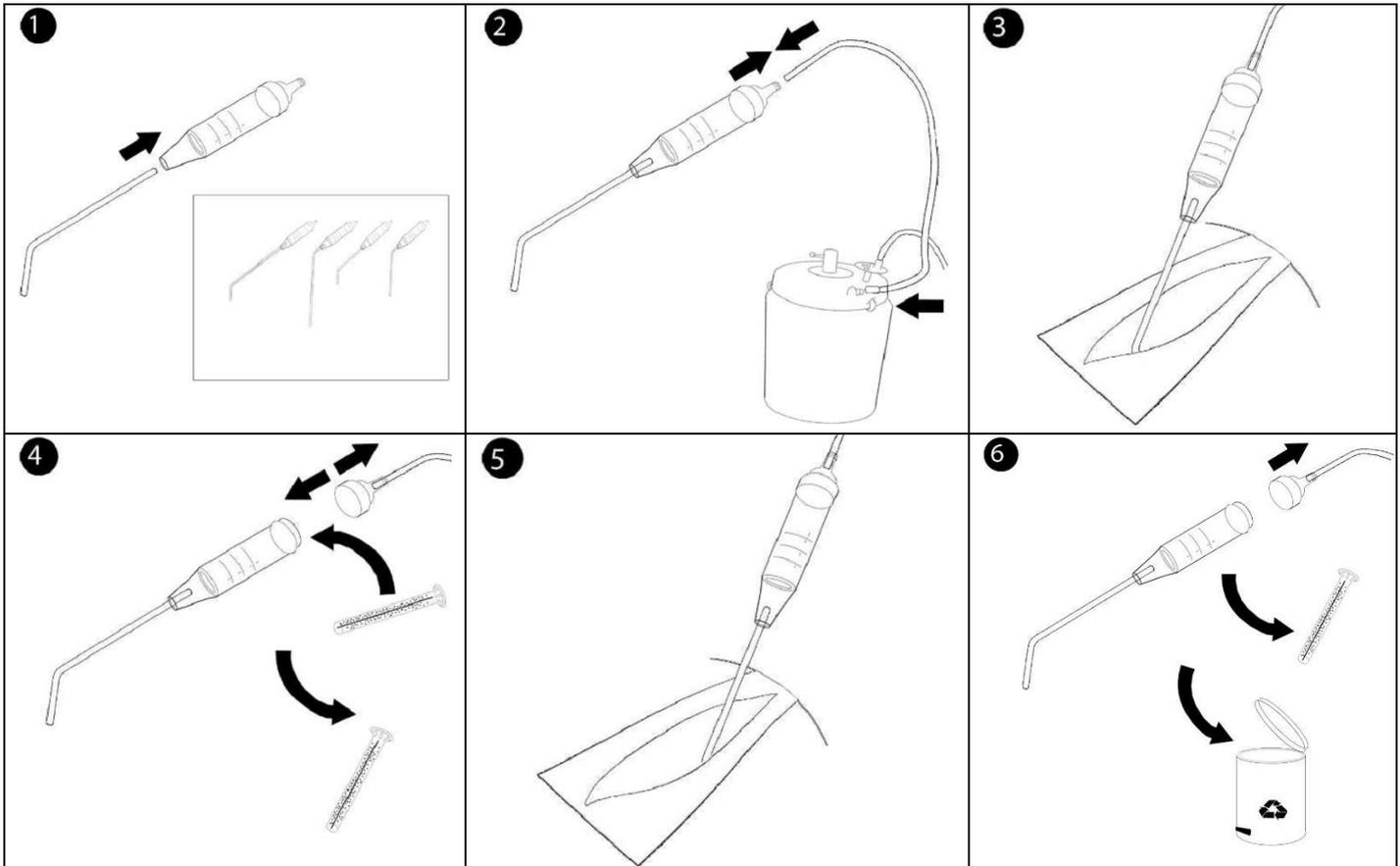

Warnings and precautions measures:

1. Always check connections tightness prior to procedure.
2. Adjust suction to the level which allows avoiding delicate tissue damage.
3. Dispose of all opened suction cannulas or suction sets no matter if they were used or not.
4. Use immediately after opening.
5. The instrument requires appropriate disposal after use in accordance with all applicable local regulations including, without limitation, those pertaining to human health and safety and the environment.
6. This product is intended for single patient and procedure use. Resterilization, reuse, reprocessing, modification may lead to serious consequences.
7. Product is intended to be used exclusively by qualified medical staff.

	 Keep dry	 Consult instructions for use	 Manufacturer	<table border="1"><tr><td>EC</td><td>REP</td></tr></table>	EC	REP
EC	REP					
<table border="1"><tr><td>STERILE</td><td>EO</td></tr></table>	STERILE	EO	 Caution, consult accompanying documents	 Do not re-sterilize	 Do not use if package is damaged	Authorized representative in EU
STERILE	EO					